

Mueller B5333 Hinged Knee Brace Deluxe

Size Chart

sizing: measure around thigh and calf

	SM	MD	LG	XL	XXL	XXXL
THIGH	17" - 18" (43 - 45 cm)	19" - 20" (48 - 50 cm)	21" - 22" (53 - 55 cm)	23" - 24" (58 - 60 cm)	25" - 26" (63 - 66 cm)	27" 28" (68 - 71 cm)
CALF	13" - 14" (33 - 35 cm)	14" - 15" (35 - 38 cm)	15" - 16" (38 - 40 cm)	16" - 17" (40 - 43 cm)	17" - 18" (43 - 45 cm)	18" 19" (45 - 48 cm)

Use Instructions:

- Loosen two rear straps.
- Loosen upper and lower sections.
- Align patella opening over kneecap, grasping both hinges to pull brace into correct position.
- Secure upper and lower sections firmly, beginning with the lower calf section.
- Secure two rear straps through loops.
- Secure two front straps for firm, comfortable fit.

Care:

- Remove hinges before washing. Hand wash in cold water with mild detergent.
Do not use bleach or fabric softener. Rinse thoroughly and air dry. If brace is used in water, remove hinges after use and rinse off thoroughly.

Caution:

- This product contains natural rubber latex which may cause allergic reactions. Constructed of a neoprene blend which may cause skin irritations. If rash develops or pain persists, discontinue use and consult a physician.